



NTIA AFHCANcart Quick Check Guide

This is a recommended quick test to verify that the NTIA AFHCANcart hardware is correctly configured and performing as expected at the end site.

Name of Tester: _____ Date: _____

Cart Location: _____

Tested	Not Tested	No Device	NOTE: Test all devices installed on your cart.
☐	☐	☐	Basic Cart Function: Cart's vertical adjustment feature, keyboard light, and steering are operational. Desktop applications include AFHCANcart client software and Zoom. Desktop shortcut to https://rooms.video/TelePatient/ is in place.
☐	☐	☐	Please log out of AFHCAN software. Shutdown telemedicine cart and restart.
☐	☐	☐	Login to AFHCAN software. Write the version of the software here (e.g. v8.4.5): _____
☐	☐	☐	Create a new TEST case.
☐	☐	☐	Select a TEST patient.
☐	☐	☐	Forms: Add any form to the case and save.
☐	☐	☐	JedMed Video otoscope: Capture and save an otoscope image to the case.
☐	☐	☐	Global Med Total Exam Camera: Capture and save a digital camera image to the case.
☐	☐	☐	Midmark Digital ECG: Capture and save an ECG waveform to the case.
☐	☐	☐	Bio-Logic Tympanometer: Capture and save a tympanogram to the case.
☐	☐	☐	Bio-Logic Audiometer: Capture and save an audiometer test to the case.
☐	☐	☐	Bio-Logic DPOAE: Capture and save a DPOAE screening to the case.
☐	☐	☐	Zoom: Add a Zoom call outside of the case, test the camera and microphone to make sure operational.
☐	☐	☐	Add a comment to the case.
☐	☐	☐	Archive the case.
☐	☐	☐	Logout of the AFHCAN software.
☐	☐	☐	Cart is fully operational. (Note any discrepancies and contact afhcansupport@anthc.org if not.)